

TECHNICAL SERVICE BULLETIN 21-008



Date Issued: October 27, 2021

Subject: VALIDATE® GC3 1300vt LDH Coverage Change

Due to the upcoming discontinuation of Ortho Vitros LDH Slides with a reportable range of 100-2150 U/L, LGC will be changing the manufacturing of the GC3 1300vt Validate kit to target the Ortho Vitros LDHI Slides with a range of 41-1000 U/L. This change will be occurring in the next lot of Validate GC3 1300vt kits, expected to be launched in January 2022.

Customers continuing to use LDH Slides will still be able to use GC3 1300vt material; however, due to reagent methodology, higher results should be expected with possible non-numerical values on level 5. If non-numerical values are recovered, a one-to-one admixture using levels 4 and 5 can be made to create a new level recovering numerical values closer to the 2150 U/L high-end of the analytical measuring range.

For questions or additional information, please contact LGC Clinical Diagnostics Technical Support department.

Contact Information: Technical Support Department
LGC Clinical Diagnostics
221 US Route 1
Cumberland Foreside, ME 04110
Phone: 1-800-377-9684 ext. 1428
+1-207-892-1300 ext. 1428
Fax: +1-207-892-2266
E-mail: msc.info@LGCGroup.com

Doc. Number: MKT-00679
Rev.: 1
Doc. Title: Customer Notification regarding GC3 1300vt LDH
Effective Date: Nov 17, 2021

Page 1 of 1